

REF X002AM0CMN

resplabs

CPAP MASK LINERS

COMPATIBLE WITH PHILIPS RESPIRONICS® DREAMWEAR® NASAL CPAP MASKS



**DISCOVER
CPAP FREEDOM:
SCAN TO LEARN
MORE**

SCAN ME



X002AM0CMN

resplabs CPAP Mask Liners - 4 Liner Pack
New

LOT



QTY 4 EACH

- Extend the liners around the CPAP mask cushion part.
- You may gradually trim the smaller liner hole if necessary for proper face and mask coverage.
- Make sure that the liners and mask opening are properly aligned.
- Avoid covering the mask's breathing ports.
- Machine wash with cold water and air dry.

For detailed information, visit:

<https://resplabs.com/pages/important-safety-information-magnetic-cpap-headgear>

For questions, call 1-888-709-7377 or email info@resplabs.com

Manufactured by: resplabs medical USA inc.

2015 N Forbes Blvd Suite 101 Tucson AZ 85745 | 1-888-709-7377 | resplabs.com

Follow your healthcare provider's advice and device instructions.

Made in Philippines.